





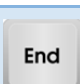




Zero Foot-Print (ZFP) Tip Sheet

Toolbar Buttons

	Study/Patient Search Availability is dependent upon site's configuration.
	Navigator
	Series Selector
	Layout Grid View_H (2H) and View_V (2V)
	Window Level Presets <ul style="list-style-type: none"> List of site defined presets
	Flip/Rotate <ul style="list-style-type: none"> Flip vertically Flip horizontally Rotate right
	Measure <ul style="list-style-type: none"> Distance Freehand (ROI)
	Image Overlay Toggle
	Cross Reference
	Presentation States <ul style="list-style-type: none"> List of Presentation States
	Cine
	Reports <ul style="list-style-type: none"> View reports and exam notes


Keyboard Shortcuts

Keyboard Shortcut	Action
Press and hold (>1 Sec) 	Scroll through the next images
Press and hold (>1 sec) 	Scroll through the previous images
	Next series
	Previous series
	Middle image in series
	First image in series
	Last image in series
	Next image in series
	Previous image in series

Tips

Grayed out tools are either not valid for the study or not configured for your site.

Click **Help**  to access the detailed operators' manual.

To **logout**, click the logout button .